

# GE CT System PM Checklist Q1

Site: <u>Green Light Imaging</u>		Date: <u>3/21/20</u>	
System ID#: <u>CT3</u>		Service Report #:	
Service Engineer: <u>Gordon Gray</u>			
Procedure	Inspected	Cleaned	Passed
COMPUTER			
Check fans/Clean/Vacuum all exhaust vent holes	✓	✓	
CONSOLE			
Check fans/Clean/Vacuum console interior	✓	✓	
Verify emergency off buttons (table/console)	✓		
Inspect Keyboard, mouse, and monitors cable connections and operation	✓	✓	
GANTRY			
Verify tilt operation	<u>N/A Gantry Locked</u>		
Inspect/Clean Mylar window	✓	✓	
Inspect/Clean All fans and filters	✓	✓	
Handling & Removal of Slip Ring Brush Debris	✓	✓	
Slip Ring Vacuuming	✓	✓	
Verify Gantry balance	✓		✓
SYSTEM			
Check operation of final scan abort push-buttons	✓		✓
Inspect Error Logs for bad channels, tube spits, and HV issues	✓		
Record current tube mAs or exams	<u>10890395.5</u>		
Verify Recon Data Path passes	✓		✓
Verify Das Tools auto test passes	✓		✓
Verify Collimator functional tests	✓		✓
Verify Collimator Aperture	✓		✓
Inspect/Clean/Check operation of table cradle drive	✓	✓	
General cleaning of all covers		✓	
Inspect high voltage connections	✓		

# GE CT System PM Checklist Q1


3/21/20 CT3

Procedure	Inspected	Cleaned	Passed
SYSTEM (Continued)			
Inspect tube cooling connections	✓		
Verify table movement	✓		✓
Verify laser light alignments	✓		✓
Verify console date and time	✓		✓
Verify console network, archive and filming queues are empty	✓	✓	
Perform Quality Assurance Scans	✓		✓
Create system state backup	created + on site		

**Notes:**

System passed all Diags + Image Performance

Service Engineer Signature:

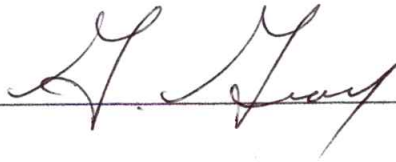

Date:

3/21/20