

# GE CT System PM Checklist

<b>Site: GreenLight CT3</b>	<b>Date: March 29, 2024</b>		
<b>System ID#:731221ct1</b>	<b>Service Report#:</b>		
<b>Service Engineer: Leonard Santos</b>			
<b>Procedure</b>	<b>Inspected</b>	<b>Cleaned</b>	<b>Passed</b>
COMPUTER			
Check fans/Clean/Vacuum all exhaust vent holes	x	x	x
CONSOLE			
Check fans/Clean/Vacuum console interior	x	x	x
Verify emergency off buttons (table/console)	x	x	x
Inspect Keyboard, mouse, and monitors cable connections and operation	x	x	x
GANTRY			
Verify tilt operation	x	x	x
Inspect/ Clean Mylar window	x	x	x
Inspect/ Clean All fans and filters	x	x	x
Handling & Removal of Slip Ring Brush Debris	x	x	x
Slip Ring Vacuuming	x	x	x
Verify Gantry balance	x	x	x
SYSTEM			
Check operation of final scan abort push-buttons	x	x	x
Inspect Error Logs for bad channels, tube spits, and HV issues	x	na	x
Record current tube mAs or exams	x	na	<b>65105737</b>
Verify Recon Data Path passes	x	na	x
Verify Das Tools auto test passes			
Verify Collimator functional test	x	na	x
Verify Collimator Aperture	x	na	x
Inspect/Clean/Check operation of table cradle drive	x	x	x
General Cleaning of all covers	x	x	x
Inspect high voltage connections	x	na	x

# GE CT System PM Checklist Q3

Procedure	Inspected	Cleaned	Passed
SYSTEM (Continued)			
Inspect tube cooling connections	x	na	x
Verify table movement	x	x	x
Verify laser light alignments	x	x	X
Verify console date and time	x	na	x
Verify console network, archive and filming queues are empty	x	na	x
Perform Quality Assurance Scans	x	na	x
Create system state backup			

**Notes:**

Added grease to the Gantry to minimize noise. Updated the Save State.

---

---

---

---

---

---

---

---

---

---

Service Engineer Signature:  Date:

